

AED Fast Facts

March 31, 2010 - #23

An E-mail Promoting Awareness of Automated External Defibrillators


Information presented to complement an established AED Program

AED “*Realty* 101- Electrodes Placements”

Help Make Saving a Life a REALITY by Knowing

Electrodes Placement Location

....Location, Location

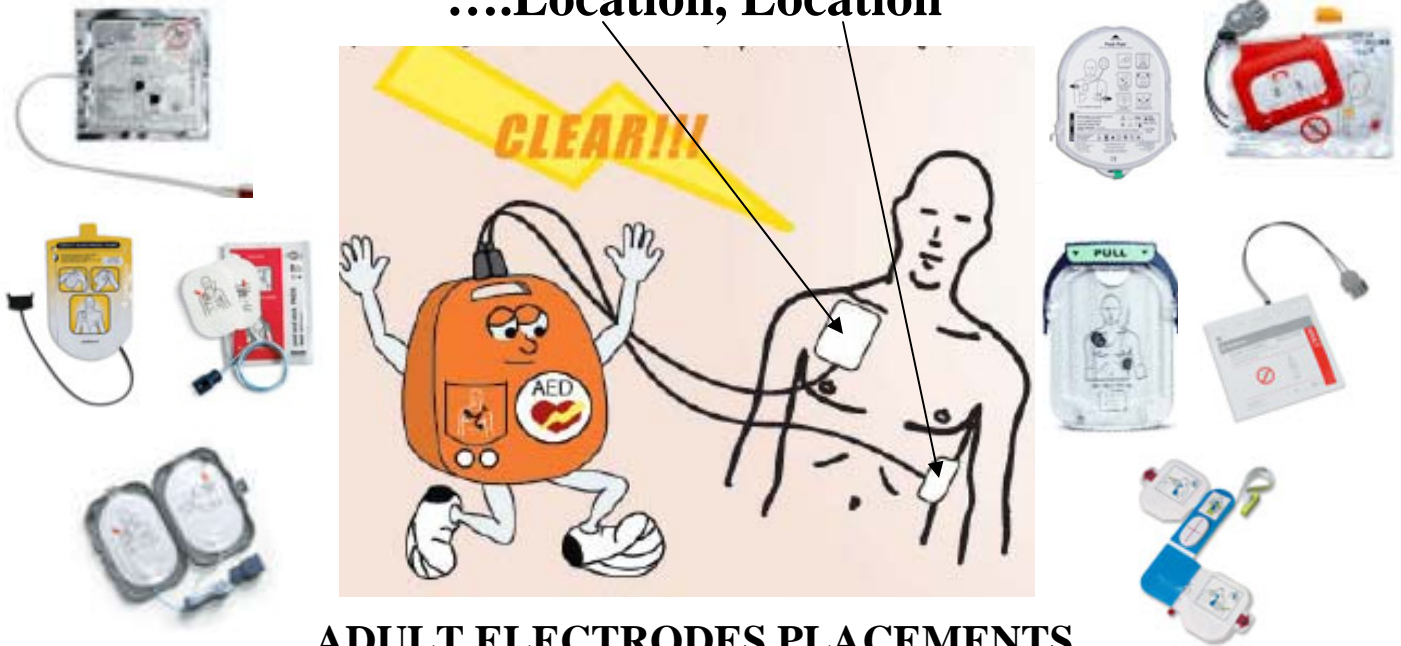

ADULT ELECTRODES PLACEMENTS

Notes: Adult Placements are on the patient's bare chest

Firmly press the electrodes on patient's skin - upper right of chest and left side as shown

Once placed, stay clear of electrodes and patient and follow AED voice prompts

Do not place pads over a medical device. Move pads at least an inch away from such devices

Child Pads placements (Age 1 to 8 years of age) vary depending on AED Model.

- Some placements are similar to the adult placements
- Some placements indicate front and back of child

Signs of Sudden Cardiac Arrest:

- Unconscious
- Not Responding
- Not Breathing or Not Breathing Normally
- Agonal Breathing or Gasping is Not Normal Breathing
- Seizures/Convulsions may occur when patient collapses

Combat Sudden Cardiac Arrest with

- Early 911
- Early CPR
- Early Defibrillation
- Early Advanced Life Support



1551 E. Lincoln Ave. Ste 166 Madison Heights, MI. 48071
800-228-3343 www.heartaed.com

Happy Spring

